# BellaBambi® original

massage to go

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Squeeze BellaBambi, put it on your skin and release your grip. Now BellaBambi is attached to your skin. You can move BellaBambi gently across your skin or leave it in place. For a more gentle application, squeeze BB less. Always listen to your body's signals and do not do anything that makes you feel uncomfortable.

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## Cleaning and Disinfection

BellaBambi can be cleaned very easily in the dishwasher or with mild soap suds. After drying, please treat BellaBambi with a suitable disinfectant for silicones (without phenol derivatives or alkyl amines (Glucoprotamin)). We recommend: Desomed® Rapid AF.

**Please note:** If BellaBambi is very dirty, please boil it like a pacifier. Use a clean pot, cover your BellaBambi completely with water and boil it for at least five minutes.

Suitable cleaning agents and disinfectants can be found on [www.bellabambi.com](http://www.bellabambi.com)

## Thank you for buying BellaBambi

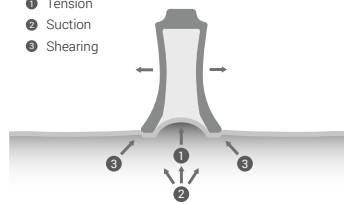
Congratulations on buying BellaBambi. This is a multi-purpose and durable product for beauty, relaxation and sports. BellaBambi is dermatologically uncritical, consists of biocompatible silicone and is produced exclusively in Germany. To have as much success and lasting pleasure with your product, we kindly ask you to read this instructions carefully.

Thank you!  
Your team from BellaBambi

## Functional Principle

BellaBambi increases blood circulation, improves lymphatic circulation and promotes flexibility. As you drag BellaBambi over your skin you can also achieve an exfoliating effect.

- 1 Tension
- 2 Suction
- 3 Shearing

## Caution

Please note our experiences with possible skin reactions when BellaBambi is permanently left in place:

	pressure mark	redness	effusion
sensitive skin	approx. 1-2 min	approx. 2-3 min	approx. 5 min
normal skin	approx. 2 min	approx. 4 min	approx. 7 min
robust skin	approx. 3 min	approx. 6 min	approx. 9 min

Art.Nr. 2829308

Bamberger Wellness GmbH  
Industriestr. 25  
D-89423 Gundelfingen  
Germany  
+49 9073 9210461  
info@bellabambi.de  
[www.bellabambi.de](http://www.bellabambi.de)

## Precautions / Contraindications

**Do not apply BellaBambi** on nipples, sexual organs, anus, eyes, eyelids, lips, mouth, in the axilla and in the inguinal region.

**Please consult with your physician before using BellaBambi if one or more of the following conditions apply:** Pregnancy, acute injuries, open wounds, inflammations, varicose veins, disposition to hematomas and ulcers caused by diabetes or vascular diseases, irritable skin disorders such as psoriasis or urticaria (hives), with blood-thinning medication therapies, tumors, Complex regional pain syndrome (CRPS), as well as any unclear or acute medical conditions.

## Intensities

Different intensities fulfill the needs of different skintypes or grades of sensitivity. Furthermore, you can slowly increase the stimulus by using different intensities.

**SENSITIVE** for soft application  
white, lemon yellow

**VITALITY** for vital application  
orange, boysenberry

**INTENSE** for strong application  
ruby, midnight blue

## Recommendations

For most applications, an oil to assist lubricity is helpful. BellaBambi can also be used with hyaluronfluid, cellulite creams or body lotions. Many suitable products can be found on [www.bellabambi.com](http://www.bellabambi.com)

Please listen to your body's signals. If you feel uncomfortable, try some lower pressure by squeezing BellaBambi more gently. Alternatively, you can try eg. BellaBambi SENSITIVE.

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Muscle oil with skin caring properties. Relaxes cramped muscles and supports the BellaBambi treatment.

Ingredients: Hypericum perforatum oil, Glycine soja oil, Arnica montana flower, Rosmarinus officinalis oil, Juniperus communis oil, Eucalyptus globulus oil, Pinus pumilio oil, Lavendula angustifolia oil, Citral\*, Citronello\*, Coumarin\*, Farnesol\*, Limonene\*, Linalool\*, Cinnamal\*, Isoeugenol\* \*) from natural essential oils

100 ml 10 ml

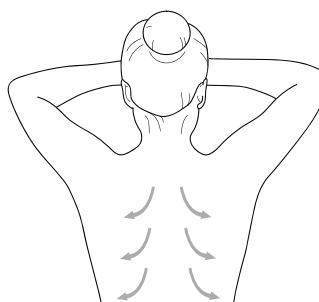
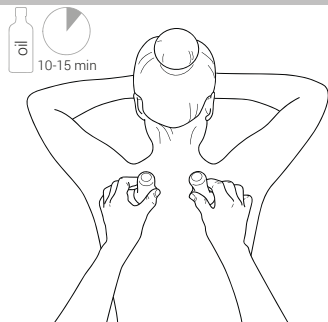
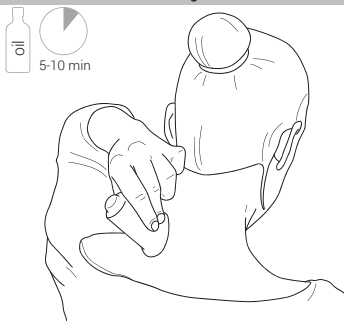
## Internet / Social Media

[www.bellabambi.de](http://www.bellabambi.de)  
[facebook.com/bellabambi.de](https://facebook.com/bellabambi.de)

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Place BellaBambi below your hairline and drag it downwards as far as possible. Afterwards relocate BellaBambi to the side and repeat. If used on another person's back, drag BellaBambi crescent-shaped outwards. Speed up or slow down according to your individual preferences.

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Place BellaBambi above your knee and drag it to your hip. Relocate BellaBambi until your entire thigh is covered. Repeat 3 times altogether.

**Tip:** Do sports before or after using BellaBambi, especially swimming or cycling. Workouts for legs and buttocks suit perfectly too.

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Place BellaBambi approx. 5 cm above your knee and drag it around your thigh without stopping. Relocate BellaBambi above and repeat the treatment until your entire thigh is covered. Repeat 3 times altogether.

**Tip:** Make sure to drink enough water or tea and eat fresh vegetables. Avoid sugar and alcohol where possible.

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Place BellaBambi below your knee, drag it to your wrist and back again. Relocate BellaBambi until your entire calf is covered. Repeat 3 times. **Variation:** Try to stretch and bend your foot while dragging BellaBambi.

**Tip:** On the sides of your knee, you can also drag BellaBambi until your thigh.

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1 **Static:** Place one BellaBambi on your forefoot, one on your arch of foot and one on your heel. Leave it there for 2-3 minutes. **Important:** Always listen to your body's signals!

2 **Dynamic:** Place BellaBambi below your toes and drag it to your heel. Continue from heel to toes in the same line.

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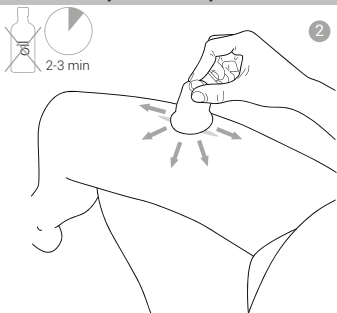
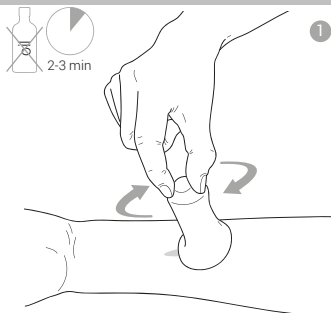
**Caution:** Please consult your doctor before using BellaBambi on scars! Be careful!  
**Important:** The scar has to be clean, dry and closed.

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
1 Place BellaBambi on the scar, grab it on the top and circuit. Alternatively, push BellaBambi in different directions for 20 seconds each.

2 Place BellaBambi and drag it in a star pattern over your scar. Please fix the initial point with your other hand, when dragging BellaBambi.

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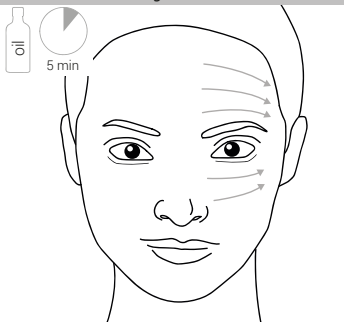
Place BellaBambi below your hairline in the middle of your forehead and drag it outward. Relocate BellaBambi 1 cm below and repeat until you reach your eyebrow. Move to your cheek and drag BellaBambi from the side of your nose to your ear.



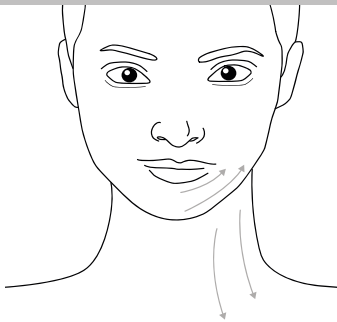
**Important:** Keep sweeping BellaBambi to minimize the risk of bruises.

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Place BellaBambi in the middle of your chin under your bottom lip and drag it over your jaw to your earlobe. Repeat until you've finished your chin. Finally place BellaBambi below your jawbone and drag it from your upper neck to your collarbone. Please do not treat your larynx.

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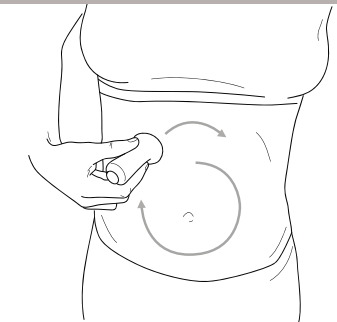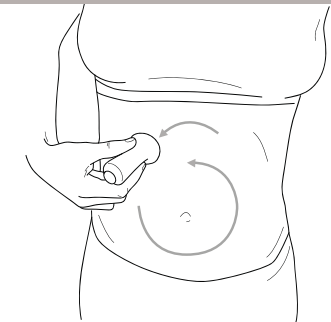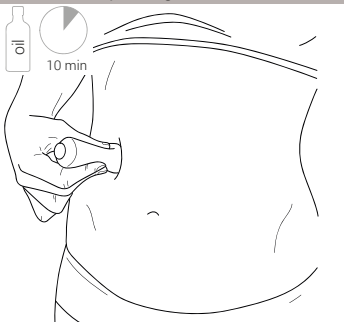Place BellaBambi below your costal arch and drag it slowly in narrowing circles around your belly button.

Afterwards do it the reverse way until you have reached the initial point. This exercise is best done lying down.

**Important:** Wind down and breathe easy.